Right Caliper Arm	Allow ¼ inch or .64CM of space between this arm and the right edge of your pet's body.
Above your pet	 HOW TO MAKE THE CALIPER 1. Tape or glue right and left caliper arm templates against cardboard. 2. Cut along the dotted lines. 3. Slightly overlap the arms and attach with a paperclip so that it looks like this:

L

 I I

J,

_ _ _

HOW TO MEASURE WITH A CALIPER


 Hold the cardboard caliper over your pet's shoulders and slide the arms so it barely touches the pet. You will need to move and adjust the paperclip to keep the arms together. Allow a ¼" space between the points of the caliper and your pet.



2. Using a ruler or metal measuring tape, measure the **inside** of the right and left arms of the caliper.



3. Note the measurement for the shoulders, and repeat for ribs and rump.

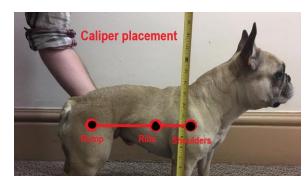

Allow ¼ inch or .64CM of space between this arm and the left edge of your pet's body.

Above your pet

Left Caliper ∆rm